

Migraine relief is as easy as 1-2-3



Ask your doctor about sTMS mini by eNeura®



Take your life back today!

It's finally here!  
The **non-drug** migraine treatment you've been waiting for...

sTMS mini™ from eNeura®  
for acute treatment and prevention of migraine

My gain

Safe, clinically effective treatment...without medication side effects



Painless treatment in less than a minute...

1 **Power**

Press the center button to turn on your device

2 **Position**

Hold the sTMS mini so it cradles the back of your head

3 **Pulse**

Press the treatment buttons to deliver a safe and therapeutic pulse

Repeat as prescribed

Use your sTMS mini™ in the morning and evening for migraine prevention, and as needed to treat acute attacks.

To learn more about eNeura and the sTMS mini™ visit our web site at [www.eNeura.com](http://www.eNeura.com)

**eNeura Customer Care**

Phone: 855-366-8355 (select Option 1)

Fax: 877-874-9584

Email: [customercare@eneura.com](mailto:customercare@eneura.com)

Hours of operation: 8:00 AM to 5:00 PM PT

In case of a medical emergency, please call 911 or contact your doctor.

**Indications**

**USA:** The sTMS mini™ is indicated for the acute and prophylactic treatment of migraine headache.

**UK:** The sTMS mini™ is indicated for the prevention and acute treatment of migraine.

eNeura Inc. 715 North Pastoria Avenue, Sunnyvale, CA 94085

© 2017 eNeura Inc. All rights reserved. eNeura is a registered trademark and sTMS mini is a trademark of eNeura Inc.

LBL-0122 REV G 1/2018

eNeura®

**\$300<sup>+</sup>** NEW PATIENT DISCOUNT

SEE INSIDE!

eNeura makes it easy and affordable to get started today

### We deliver the sTMS mini™ to your door!

- As a new eNeura patient, you will automatically receive a \$300† discount on your first prescription (regular price is \$750 for 3 months)
- No insurance prior approvals or letters of medical necessity required.

### eNeura experts at your service

#### Clinical Education Specialists

You will receive a personal orientation call from one of our Registered Nurses. They will provide guidance on using your sTMS mini™ and support you in monitoring your progress.

#### Access Consultants

In a world of high deductibles and copays, insurance coverage is complicated. Our professionals will work with you to explore funding options that best meet your needs including insurance coverage, HSA, FSA, HRAs, and Care Credit and eNeura discounts.

#### Customer Care

This team will assist you with prescriptions and refills.

†Terms of offer subject to change without notice

An effective treatment without the risk of intolerable and often disabling side effects

	sTMS mini™	Medications	OnabotulinumtoxinA
<b>FDA Clearance or Approved Migraine Indication</b>	Prophylactic and Treatment of Acute Attacks	Prevention	Prevention of Chronic Migraine (>15 days/mo)
<b>Efficacy</b> % of Patients with more than 50% reduction in Headache Days	46%* After 3 months of treatment	41%**	47%*** After 6 months of treatment
<b>Potential Side Effects Patients May Experience</b>	Momentary lightheadedness and ringing in the ears at time of treatment	<ul style="list-style-type: none"> <li>Slowed Thinking</li> <li>Fatigue</li> <li>Dizziness</li> <li>Numbness and tingling in hands and feet</li> <li>Birth Defects</li> <li>Cardiovascular issues</li> </ul>	<ul style="list-style-type: none"> <li>Injection site discomfort</li> <li>Neck pain</li> <li>Eyelid drooping</li> <li>Weakness or partial loss of movement in the face</li> <li>Allergic reactions</li> <li>Spread of toxin effects (loss of strength and all over muscle weakness)</li> </ul>

Sources: \* Starling A, et al. (2017). A Multicenter, Prospective, Single Arm, Open Label, Post-Market, Observational Study to evaluate the use of sTMS in reduction of Migraine Headache (ESPOUSE Study). Data presented at the American Academy of Neurology's Annual Meeting in Boston, MA. \*\*Macedo, A. et al. (2008). Placebo response in the prophylaxis of migraine: A meta-analysis. European Journal of Pain. \*\*\* Preempt I/II. www.botoxmedical.com/ChronicMigraine/Efficacy/ClinicalResults. Retrieved 10/2017.

### How sTMS works



sTMS mini™ from eNeura is an ingenious application of single pulse Transcranial Magnetic Stimulation (sTMS), used in neurology clinics for over 40 years—but never before available in a portable, easy-to-use device that allows patients to treat themselves.

Sources: Andreou AP, et al. (2016) Transcranial magnetic stimulation and potential cortical and trigeminothalamic mechanisms in migraine. Brain. Barker AT and Shield K. (2017) Transcranial Magnetic Stimulation: Basic Principles and Clinical Applications in Migraine. Headache: The Journal of Head and Face Pain. Dodick DW, et al. (2010). Transcranial Magnetic Stimulation for Migraine: A Safety Review. Headache: The Journal of Head and Face Pain.

- sTMS is the only safe, non-invasive neuromodulation technology designed to work at the source of migraine - the cortex and deep brain structures which process pain and other signals.
- This innovative technology uses a magnetic pulse to deliver a mild electric current through the skull and into the outer layer of the brain.
- The magnetic pulse modulates the electrical environment of the nerve cells and reduces the hyperactivity associated with migraine.
- Recent scientific and clinical studies have shown that preventive or pre-treatment with sTMS may block or prevent the onset of an attack by suppressing or reducing the onset of hyperactivity associated with migraine.