

# Welcome to the future of migraine care

## SAVI™ Smart neuromodulation technology

Take your life back - enjoy pain free, clear blue days

- Stop migraine attacks
- Prevent headache days
- Automatically track your treatment progress
- NO medication side effects

FDA Cleared for the acute and preventive treatment  
of migraine in adults and children (12 and older)

[Quick Start Patient Guide](#)



SAVI by eNeura



I am just ecstatic about my results. This technology is truly life-changing for me...The Customer Care and Nursing Support have been outstanding and made getting started so easy.



—53-year-old female



## SAVI Smart neuromodulation technology + your personal support team



### Clinical Education Specialists (Registered Nurses)

- Provide guidance on using SAVI
- Help you use your SAVI Online Migraine Diary
- Support you in monitoring your progress
- Assist you in communicating with your prescriber



### Customer Care

Available to help you with any questions about:

- Caring for your SAVI
- Prescription renewals and deliveries
- Traveling with your SAVI

## What's in the box?

This package contains everything you need to get started with your SAVI—a safe, clinically proven migraine treatment.



## Your SAVI

Battery-powered,  
rechargeable device

### SAVI Patient Quick Guide

For the complete guide to the SAVI,  
please visit our website at [www.eneura.com](http://www.eneura.com)



### Battery charger (AC adapter)

For recharging your SAVI  
battery

We recommend you  
charge your device  
daily.



### Carrying bag

For carrying your SAVI  
with you everywhere you  
want to go.



## Get to know your SAVI™

The buttons and displays on the device are easy to understand.

For migraine management made easy, your SAVI includes Smart Technology to automatically download treatment history and prescription renewals.



The power button will blink blue during updates and will shutoff automatically when complete.

# Symbol Key



## Prescription status indicator

Located on the far left of the top panel. Confirms that a valid prescription is active. Shows the status of the prescription programmed on the device. Please see page 12 of this patient guide for details.



## Contact eNeura® customer care

Action required. Call eNeura customer care for assistance:

**US:** 1-833-499-9300 option 1

**UK:** +44 020 3014 8939, option 1



## Temperature warning

The temperature of the device is not in the range for safe use. The temperature range for use is between 16° C (60° F) and 40° C (104° F).



## Power button

Located in the center of the top panel. Press the Power button to turn the device on and off. When the Power button shines white, the device is on and ready. When the power button shines blue, it is connecting to the cell network.



## Treatment progress indicator

LED lights around the Power button show status as the device prepares for treatment. When the oval around the Power button is fully illuminated, the device is ready to use.



## Lock indicator

Located directly to the right of the Power button. Indicator is visible when device is turned on *and* the security lock switch is enabled. The security lock switch is located on the back of the device.



## AC adapter

Located to the right of the Power button. Indicator lights up when the AC adapter (battery charger) is connected to the SAVI.



## Battery capacity

Located to the far right of the Power button. Indicates how much power is left in the battery and whether or not there is enough to allow for treatment. All 4 bars will illuminate when the device is fully charged. As the power level lessens, the number of illuminated bars will decrease.




## Treatment buttons

Located on the right and left edges of the top panel. Press one or both buttons to deliver a treatment.

SAVI Smart Technology automatically tracks your treatment history, no apps required!

## How to track your treatment progress

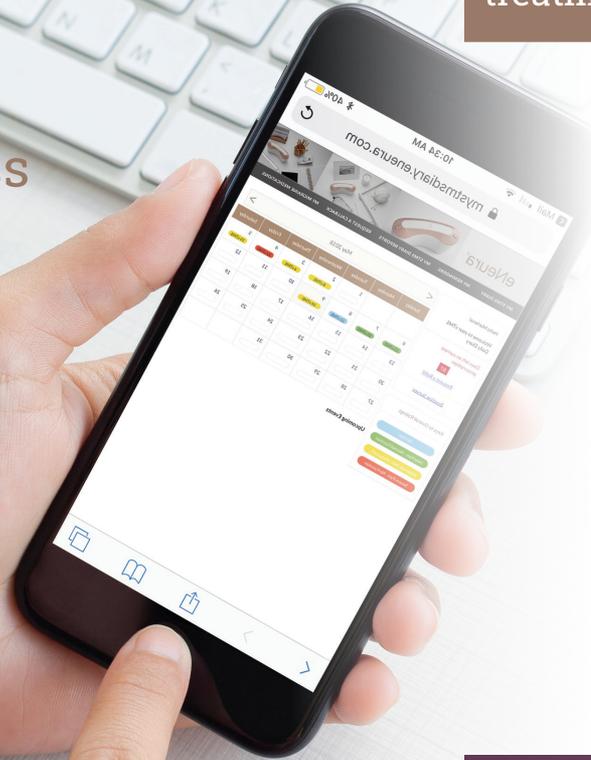

### Getting started with the SAVI™ Online Migraine Diary

You will receive an email to set up your Online Migraine Diary.

**Why keep an online diary?** It gives you the power to manage your migraine, track your treatment progress, and secure insurance coverage.

#### We've made it easy and secure.

- Opt-in for automated daily text or email reminders
- HIPAA compliant Health Cloud
- No personal health information transmitted



Access your My eNeura Diary at [mystmsdiary.eneura.com/patient/s/login/](https://mystmsdiary.eneura.com/patient/s/login/)

## How to set up and start your diary

- 1 **Click** on the link in the email you receive from **My eNeura Diary**
- 2 **Follow** the prompts to create your password
- 3 **Save** link on login for easy future access
- 4 **Open** your diary
- 5 **Enter** your migraine history

## How to use your diary

- 1 **Click** link from text or email
- 2 **Rate** your day
- 3 **Enter** any meds used that day
- 4 **SAVI** automatically downloads your treatment history for the day



It only takes a few seconds to update your diary every day— track your progress and share with your doctor.



This is the first headache diary that does the work for me. I love seeing the improvement, and my doctor does, too!



—46-year-old female



For the first time in years, I have headache-free days.



—48-year-old female



#### Lock ON/OFF switch

Located behind the accessory port door, on the left. Slide the switch to the left or right to unlock or lock the device.



#### Battery charger port

Located behind the accessory port door, in the center. Recharge the batteries by plugging the AC battery charger into this port. **We recommend you charge your SAVI™ nightly, like you charge your cell phone.**



#### Volume ON/OFF switch

Located behind the accessory port door, on the right. Slide left or right to turn the sound on or off.



#### Micro SIM Rx card port

Located on the back of the SAVI at the bottom of the accessory door. Insert the micro SIM Rx card into this port (SIM may be needed in areas where cellular network access is limited.)



SAVI - back of device view



Your device will ship in "Lock On" mode.  
Slide the switch to the left to unlock the device.

## How to use your SAVI™

You and your healthcare provider can develop a treatment plan that works for you. Your prescription includes plenty of pulses for daily preventive treatment and for treating migraine attacks - treat as early and as often as needed.

### Sample Treatment Plan:


**Daily preventive treatment:** 4 pulses twice daily

**Acute Treatment:** 3 pulses at the first sign of the migraine attack - the earlier, the better. If needed, treat again every 15-30 minutes until migraine attack subsides.

Delivering a treatment is easy and takes less than a minute.


### ① POWER

- Press the center button to turn on your device
- When the oval racetrack around the power button is fully illuminated, the device is ready to position and treat.

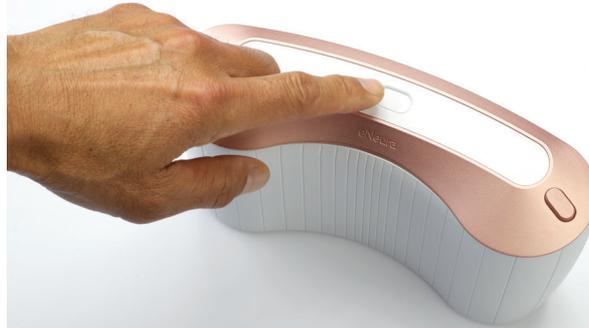

### ② POSITION

- Place your thumbs below the side button at the same angle
- Lift the SAVI over your head so it cradles the back of your skull
- Slide your thumbs over the treatment buttons on the sides when you are ready to deliver a pulse

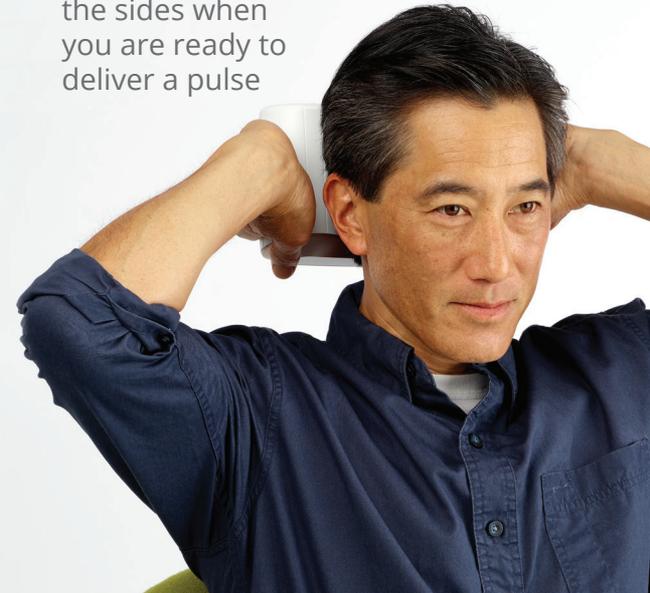

### ③ PULSE

- Press one or both treatment buttons to deliver a safe and therapeutic pulse
- Once you have pressed the button, the treatment is instantaneous and you may remove the device from the back of your head

☰ Repeat as prescribed.



Personalized treatment for migraine attacks puts you in control - treat as early and often as needed.



Watch a demonstration video at [eNeura.com](https://www.eNeura.com)

## How to renew your prescription



**Fifteen days before your prescription expires,** the prescription status indicator on your device will display an **orange warning light**.

**Rx** If the indicator glows orange, your prescription will be expiring in less than 15 days.

**Rx<sup>o</sup>** If the indicator displays with a zero next to it, the prescription has expired and the device has been disabled.

When you see this orange warning light:

- Send a renewal request via your Online Diary, or
- Contact Customer Care to renew your prescription

Email: [customer care@eneura.com](mailto:customer care@eneura.com)

Toll Free: 1-833-499-9300, option 1

- Once you pay the renewal invoice, your new prescription will be sent to your device via cell network.

## Safe usage and contraindications

Your doctor prescribed this device for your use only.

The SAVI™ generates a powerful single-pulse magnetic field as part of delivering therapy.

Treatment is very well tolerated. A small number of patients reported mild light headedness and tingling during treatment.

**CONTRAINDICATIONS:** This device should not be used in patients who have metals, conductive materials, or metal-containing implants in their head, neck, or upper body.

Dental implants, fillings, or other dental appliances are safe and are not affected by the device.

See the Instructions for Use for a complete list of contraindications and warnings at: [www.eneura.com/prescribing-resources](http://www.eneura.com/prescribing-resources)

Keep out of reach of children.

### INDICATIONS:

The SAVI is indicated for the acute and preventive treatment of migraine headache in adults and children ages 12 or older.



## To begin your journey to migraine relief

complete the form on the back of this guide. Your Clinical Education Specialist is available to assist you.



## US Customer Care

**Phone:** 1-833-499-9300, option 1

**Fax:** 1-877-264-1818

**Email:** [customer care@eneura.com](mailto:customer care@eneura.com)

### Hours of operation:

8:00 AM to 5:00 PM PST

Monday – Friday

In case of a medical emergency, please call 911 or contact your doctor.



To learn more about eNeura and the SAVI, visit [eNeura.com](http://eNeura.com).



More clarity, less  
"fuzzy" days,  
fewer headache  
days per month.



—54-year-old female

Clinical Education Specialist name: \_\_\_\_\_

Phone number: \_\_\_\_\_

Best way to contact: \_\_\_\_\_ Best time to contact: \_\_\_\_\_

### Diary access

Username: \_\_\_\_\_

Password: \_\_\_\_\_

### Headache history

# of headache days per month: \_\_\_\_\_

#### Baseline # of headache days per month

On average how many days per month do you have: (the total days should be 30)

No pain \_\_\_\_\_

Mild headache - Normal Function \_\_\_\_\_

Moderate Pain - Reduced Function \_\_\_\_\_

Severe Pain - No Function \_\_\_\_\_

### Medications

Preventive: \_\_\_\_\_

Acute: \_\_\_\_\_

### sTMS prescription

Preventive: \_\_\_\_\_

Acute: \_\_\_\_\_

### Describe what success with your SAVI looks like to you

(for example: attending sporting events or concerts, not missing work or school, etc.)

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

### Next Clinical Education Specialist appointment:

Date: \_\_\_\_\_

Time: \_\_\_\_\_

### Renew your prescription before it expires

Prescription expiration date: \_\_\_\_\_



If your Prescription Status Indicator on your SAVI turns orange, your prescription expires in less than 15 days. Please contact Customer Care to order a new prescription card.